

Porter Nasal Hood Capnography Kit

Part Number: 5054-CAP-KIT

Includes: Hole Punch and 25 Single-Use-Disposable Luer Locks

Retail Price: \$75

Part Number: 5054LUERLOCK

Includes: 25 Single-Use-Disposable Luer Locks

Retail Price: \$25



Instructions For Use:

1. Punch a hole in mask. If using Porter Double Mask – punch a hole in both the outer hood and inner liner.
2. Remove hole from mask (and liner).
3. Insert plastic luer lock.
4. After use – dispose plastic luer lock. Single Use Only.
5. If using Porter Autoclavable Hood Kit – once hole is punched you will always need to use with a luer lock. If not monitoring capnography the luer lock must be capped (included) to avoid N2O from leaking through.