


1540-1 & 1540-2 ASSEMBLIES

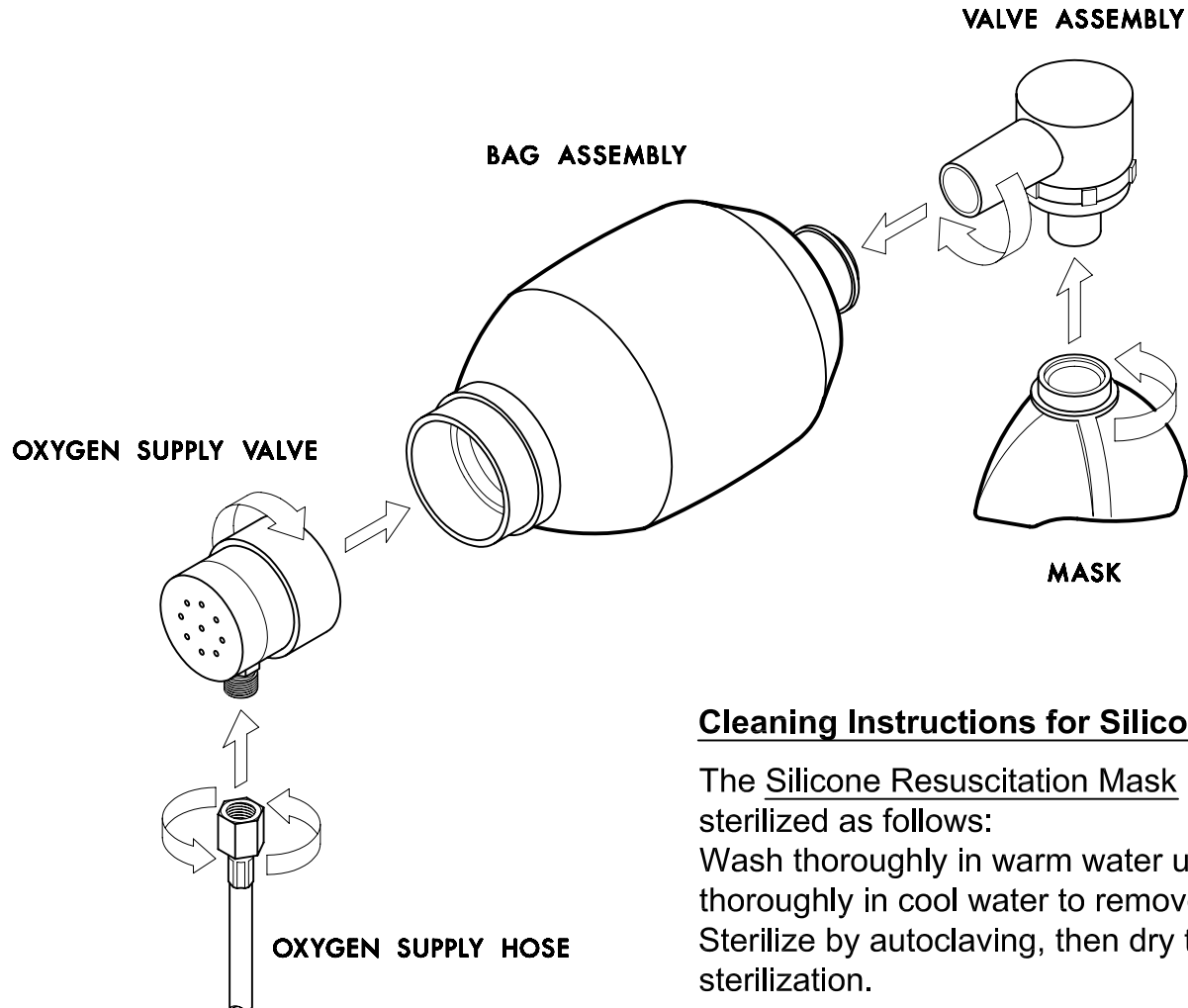

Cleaning Instructions for Silicone Resuscitation Mask Only

The Silicone Resuscitation Mask should be cleaned and sterilized as follows:
Wash thoroughly in warm water using a mild detergent. Rinse thoroughly in cool water to remove all traces of the detergent. Sterilize by autoclaving, then dry the mask thoroughly after sterilization.

NOTE: Refer to Instruction and Maintenance Manual for cleaning techniques of other component parts.



PORTER

Parker Hannifin Corporation
Porter Instrument Division
245 Township Line Rd.
P.O. Box 907
Hatfield, PA 19440-0907 USA
(215) 723-4000 / fax (215) 723-5106

FM-968 REV. A